

Quick Start Guide



NO UNAUTHORIZED MODIFICATIONS
Do not make any changes or modifications to this product without the prior express written approval of Lab Sensor Solutions Inc. Any changes or modifications made without express written approval could void the user’s authority to operate this product.

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

© 2015 Lab Sensor Solutions, Inc.
All rights Reserved.

T-Tracks™ is a Trademark of Lab Sensor Solutions, Inc.


Corporate Office: 831 Jefferson Avenue, Redwood City, CA 94063
Telephone: +1 (650) 713-2953 **Email:** forinfo@lsstracks.com **Website:** lsstracks.com



Thank You

Your Lab Sensor Solutions T-Tracks™ evaluation kit includes 5 sensors to track the temperature and location of your clinical samples in transit.

With T-Tracks, your lab can cost-effectively monitor clinical samples in real time. T-Tracks is a complete solution enabling you to know, at all times, the location and temperature of samples in transit.

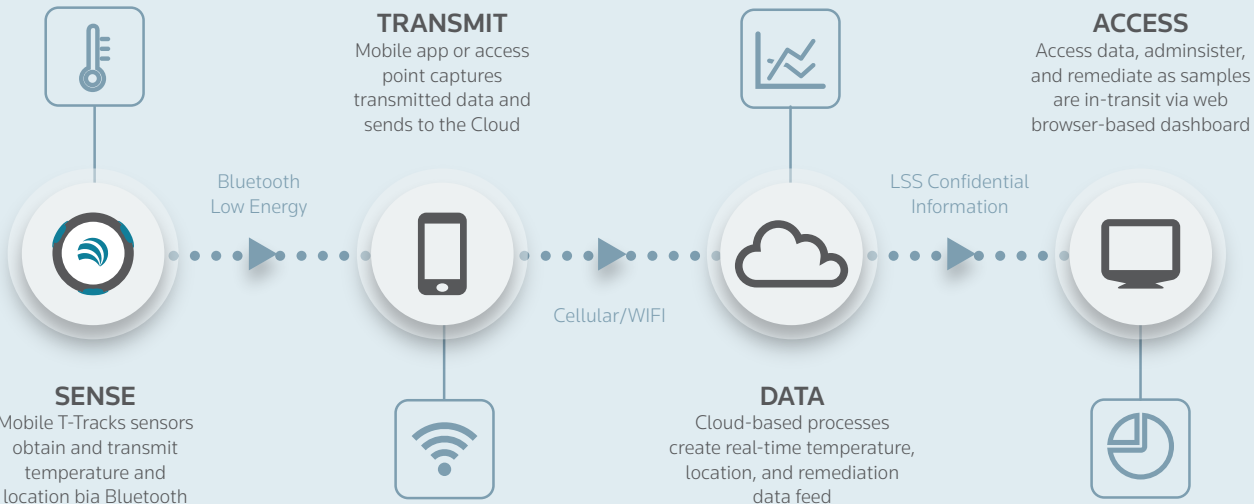

FOR SUPPORT: Call: (650) 713-2953, 6am to 6pm Pacific Time or Email: support@lsstracks.com

Getting Started

You will need

-  **T-Tracks™ Kit**
-  **Desktop, Laptop, Tablet, or Smartphone**
(with Internet access)
-  **Android Phone**
-  **Cooler or other item(s) you want to monitor**


1

Create Account

You'll need to sign up for an account online to get started.

Open a Web browser.
(You can use a Desktop, Laptop, Tablet, or Smartphone.)


Go to the site:
<http://my.lsstracks.com>
There you'll see instructions for registering a new account.

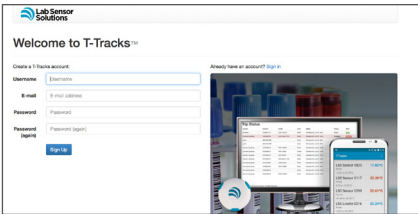

After signing up for a new account, you will receive an email with the Subject: "Please Confirm Your E-mail Address".

Click on the link to verify your email address.

Click the "Confirm" button.

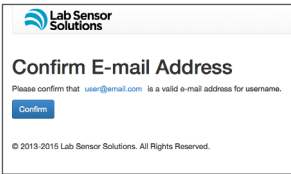

2


Activate Sensors

This step will activate your 5 new sensors.


Login using your my.lsstracks.com user name and password.



Enter your activation code:

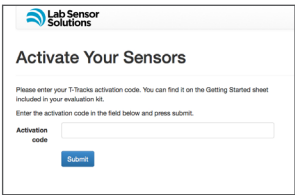

Your T-Tracks sensors have been activated.

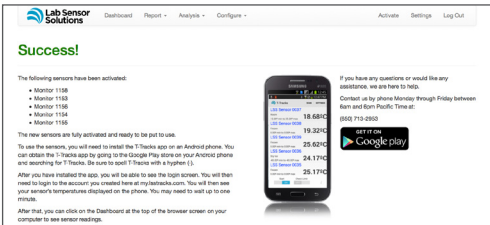

The next step is to install the app on your Android phone.

3

Install App


Now, find the Play Store app on your Android phone and open the app.

Search for T-Tracks .
Be sure to spell "T-Tracks" with a hyphen "-".

Tap on the T-Tracks App.




Tap on the "Install" button.

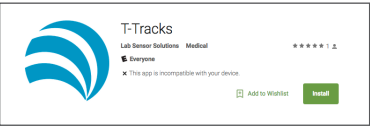

Open the T-Tracks App.

Login using your my.lsstracks.com user name and password from step 1.

The T-Tracks Screen will now show a list of the sensors that came in your kit. (This make take a minute.)


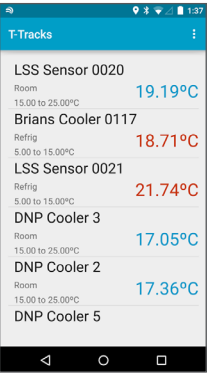


If you do not see a list of sensors on your phone screen, please contact us:
+1 (650) 713-2953, 6am to 6pm Pacific Time.

4

Call To Customize

My.LSSTracks.com, click on "Dashboard" from the navigation menu.

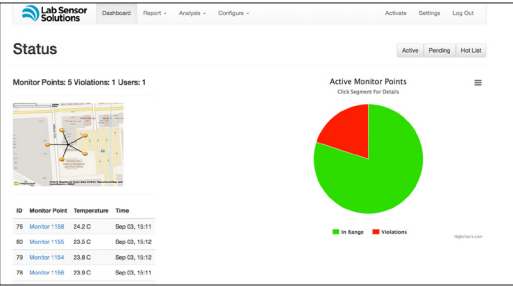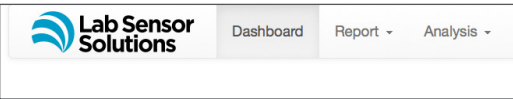

Please call us to complete your installation at this number:
+1 (650) 713-2953

(We are available from 6am to 6pm Pacific Time)