


Instructions for Programming the PAM Transmitter

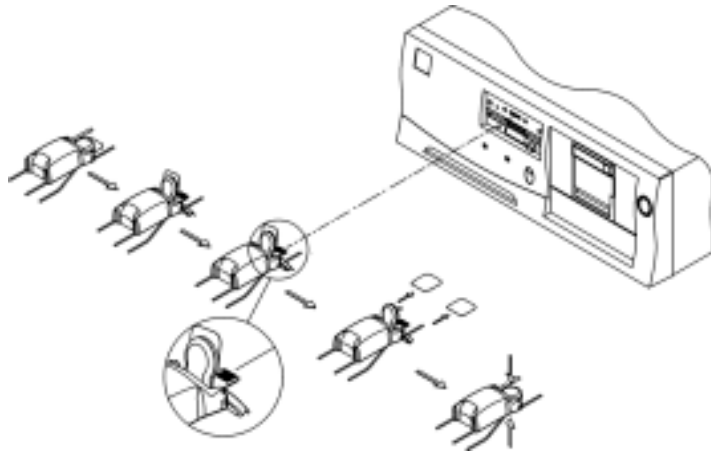

- Step 1. While still in package, expose transmitting head of transmitter.
- Step 2. Fold back both sides of foam programming head to expose programming tab. Do not remove release liners.
- Step 3. Insert tab into programming port on front of Vision base (see Illustration).
Once transmitter is inserted in programming port, system will automatically provide instructions for finding correct screen programming.
- Step 4. Follow on screen programming instructions.
- Step 5. After transmitter is programmed, remove from programming port.
- Step 6. Write patient ID onto device in the space provided.
- NOTE: To maximize battery life, DO NOT perform Steps 7 and 8 until you are ready to begin patient monitoring.**
- Step 7. While still in package, remove release liners on both halves of programming head.
- Step 8. Close and press the foam programming head together to activate transmitter.
- Step 9. Remove transmitter from package, apply to patient.



WARNING: Once programming of the transmitter is complete, the patient's ID MUST be written on the transmitter to ensure the vital signs correlate to that particular patient.