

Band attachment instructions

Once the signaling device has been activated, follow these instructions to attach the band.

⚠ WARNING ⚠

The WanderGuard signaling device is designed for wrist placement. Contact WanderGuard Customer Service at 800-824-2996 for special instructions if ankle placement or other non-standard use is intended.

- 1) Slip the band through the slot in the signaling device, Fig. 2. Be sure the snaps are positioned as shown.
- 2) Place the band around the resident's wrist with the signaling device atop the wrist, like a wristwatch.
- 3) Slip the tips of two fingers between the resident's wrist and the band to gauge for proper fit, Fig. 3. Connect the band onto the rivet as shown.
- 4) Check to be sure the band

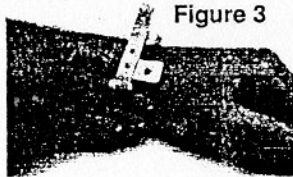


Figure 3

- will not slip over the resident's hand, and that it is loose enough for comfort and to allow for washing underneath.
- 5) Once correct band adjustment is made, double the remaining portion of the band over the rivet, then secure it with the locking tab, Fig. 4.
 - 6) Make certain the locking snap is secure, then trim off the excess band.

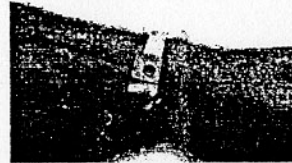



Figure 4

⚠ WARNING ⚠

Make regular inspections to confirm the band is in place. Test the device daily per manufacturer's instructions and record the results in the resident's records. Failure to do so could result in injury or death to a person in your care.

DRAFT Operating Instructions

90-Day Signaling Device
Model 32013
0163-061

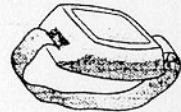

WanderGuard
DEPARTURE ALERT SYSTEM

A DIVISION OF
Senior Technologies
P.O. Box 80238 Lincoln, NE 68501

To reorder devices call 800-235-8085

www.seniortechnologies.com



32013 Activation

DRAFT

The WanderGuard signaling device is not compatible with the FirstQ departure alert system. Before activating, check to make sure the signaling device you've received does not have a black back.

The WanderGuard signaling device, Model 32013, is "off" when shipped to you. To turn "on" (activate) the signaling device:

- 1) Turn "on" (activate) the device. Use a ballpoint pen or similarly pointed object to push in the small circular disk on the side of the device until it stops, **Fig.1**. If the disk does not appear nearly flush with the side, call 800-824-2996.
- 2) Place the wristband through the slot on the 32013 with the rivet facing toward the front of the device, **Fig. 2**. DO NOT fasten snaps on wristband yet.
- 3) Check the device for proper operation by standing a minimum of three feet from the door. Open the door. The alarm should sound. Test the device in accordance with the instructions in the WanderGuard User Manual, on the signaling device tester, or see "Testing".
- 4) Note the date on the back of the device. This is the last day the device can be activated to provide approximately 90 days of useful life. Record the date in the resident's records.
- 5) Place the device on the resident's wrist (see opposite page for instructions).

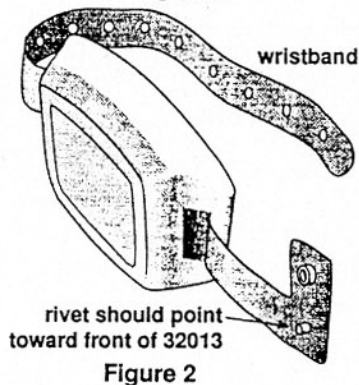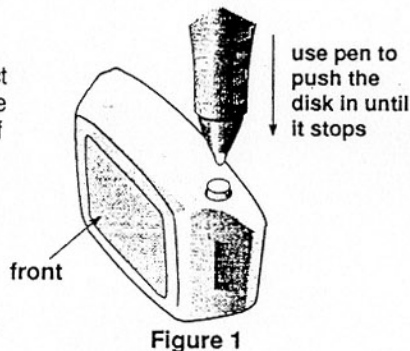

Testing

No other signaling device should be within 6' of the device you're testing. Check for frayed, loose, or missing bands at every opportunity. Replace signaling devices promptly, according to established protocols.

Testing with a signaling device tester

- 1) Point the tester toward the signaling device, and hold it within two feet of the signaling device. No other signaling devices should be within six feet of the tester.
- 2) Press the button. The TEST light will flash red or green for three seconds, then turn green to indicate the signaling device is functioning properly.
- 3) If the tester's TEST light does not turn green, refer to WanderGuard signaling device tester instructions.

Testing without a signaling device tester

- 1) Move the signaling device within three feet of a properly operating WanderGuard monitor.
- 2) Open the door (or activate a passive infrared sensor). The alarm sounds if the signaling device is active.
- 3) Move signaling device away from the monitored area and reset alarm.

⚠ WARNING ⚠

Test daily as part of a regular safety protocol in accordance with the manufacturer's instructions. Failure to do so could result in injury or death to a person in your care.

⚠ CAUTION ⚠

Changes or modifications not expressly approved by the party responsible for FCC compliance could void the user's authority to operate the equipment.

This device complies with Part 15 of the FCC rules. Operation is subject to the following two conditions:
1) this device may not cause harmful interference

and 2) this device must accept any interference received, including interference that may cause undesired operation.

US Patent 4,682,155
US Patent 5,057,824
Canada Patent 1,258,693
Canada Doc Cert. 2459 101 797

DOC-03-32013-AA
FCC# LA5SWSD005

▲WARNING▲

The WanderGuard signaling device is not compatible with the FirstQ departure alert system. Check the signaling device you have received to make sure it does not have a black back on the case. If your signaling device has a black back, call Customer Service at 800-824-2996 immediately.

TEST BEFORE USING

The components inside this signaling device were carefully tested prior to sealing the case. However, it is impossible to pretest the activation switch (see Activation section of these instructions.) It is therefore important, after activation, to test the signaling

device before putting it into use. Thereafter, test the device daily and record the results in the resident's records. If a signaling device fails to activate call WanderGuard Customer Service at 800-824-2996.

Do not use more than one signaling device on a resident. Two signaling devices, if brought very close together, may not be received by the door monitor and the alarm will not sound.

WanderGuard equipment should be operated only by trained personnel who are thoroughly familiar with the procedures outlined in the User Manual.

Failure to comply could result in injury or death to a resident.

Sudden impacts could damage components and, in rare instances, the battery could fail prematurely. The device should be tested daily (and after impact, if applicable) to check its service. Failure to regularly test could result in injury or death to a resident in your care. Record all test results in the resident's records.

Do not place the signaling device on or next to metal such as wheelchair frames or jewelry. Metal could interfere with the signal sent to the door monitor.

Approximate 90-Day operating life

Due to variances in manufacturing dates by the battery manufacturer and by WanderGuard, model 32013 has up to a six month shelf life from the manufacturing date. **Model 32013 should be activated by the date stamped on back of the case to provide approximately 90 days of useful life.**

Manufacturer Warranty

Please see the WanderGuard user manual for specific warranty information or call WanderGuard Customer Service at 800-824-2996.